


# ColloSkin

## Sterile Collagen Sheet in Wet Form

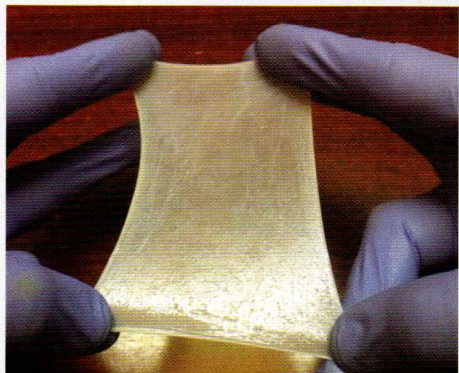

## Introduction

Colloskin is a sterile collagen in sheet form. Colloskin's use has been widely accepted biological wound dressing especially for burns. It is highly effective for patients including those with HIV and Hepatitis infections.

## Advantages

- Widely accepted biological wound dressing especially for Burns.
- Single application on Clean wounds.
- Stimulates healthy Granulation.
- Rapid & Effective wound healing.
- Quick and firm adherence.
- Prevents Infection.
- Instant pain relief.
- Allows skin to breath.
- Preserves local heat.
- Promotes Epithelisation.

## Indications

- Ideal for Non-Infective 2nd Degree Burns.
- Superficial & Deep Dermal Burns.
- For 3rd Degree Burns as a temporary cover.
- Traumatic loss of skin cover.
- Temporary wound cover in major open fracture wound(s).
- Skin donor sites

## Available Sizes

- 5 X 5 cms
- 10 X 10 cms
- 10 X 25 cms
- 15 X 30 cms

## Directions Of Use

- Thoroughly clean and denude areas to eliminate external contamination.
- Thoroughly debride infected wound.
- Take out the collagen sheet from the preserving medium and squeeze it to remove the excess preserving medium adhered to the colloskin.
- Take a trough and fill it with sterile saline water and rinse the colloskin thoroughly 2-3 times to remove the alcohol(preserving medium).
- Apply the sheet firmly on the wound and remove the air bubbles trapped if any for effective adhesion.
- For best results avoid second dressing over colloskin.
- As epitheliasation occurs, colloskin sheet peels off.
- Colloskin promotes healthy granulation in deep wounds for early grafting.
- Repeated dressing not required unless colloskin is rejected due to underlying pus or complication.
- For skin donor sites, Colloskin must be applied after achieving satisfactory haemostasis.
- For chronic skin ulcers, 2 to 3 applications of Colloskin or more in 2-3 day intervals must be repeated after thorough cleaning each time, thus enhancing healing.
- In cases where the adhered Colloskin has to be removed from the wound, a saline, moist pad can be applied over Colloskin for a few minutes to facilitate easy removal.